## 1. Allgemein:

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## 1. General terms:

- 1.1. Name of the product:** FINALE Wax smoother article no. 507-0000  
**1.2. Delivery form:** 1 bottle, 20ml transparent liquid with brush for application  
**1.3. Manufacturer:** YETI Dentalprodukte GmbH  
Industriestraße 3 - D-78234 Engen/Germany  
Tel.: (49 )07733 -9410-0 Fax: (49) 07733-94 10 22  
**1.4. Storage :** upright and dry at a temperature of + 15 - +30° C  
keep away from flammable sources

## 2. Use according to determinations:

- This wax smoother is a special varnish that sticks to wax.
- To be used particularly to seal and smoothen the occlusal area of waxworks for crown and bridge technique, to reinforce points of contact, the basal area or any other uneven wax surfaces.
- Leaves a layer of 2-3 my when applied with the brush.

**Any claims on warranty or liability will not be accepted if the product is not used according to determinations !**

## 3. Risks and Danger during application:

- ⇒ Ensure that unqualified persons (e.g. children) do not come into contact with the product. Only skilled authorized personnel is entitled to work with this product. To be used according to operating instructions.
- ⇒ Keep away from flammable sources and do not smoke during application !
- ⇒ To be used at undisturbed working places with good ventilation.
- ⇒ Modification or mixing of the product with other substances leads to the loss of liability and warranty.
- ⇒ Liquid is flammable and should be protected from open flames.

## 4. Operating Instructions:

- 4.1. Preparation:** Open the bottle after ensuring that it is intact.  
**4.2 Application:** Apply a thin layer onto the object, repeat if necessary.  
**4.3. After Application:** Allow drying for aprox. 1 minute and proceed with embedding immediately!

**5. Disposal:** According to local current regulations.

**6. Durability:** Consumable goods should be used rapidly after opening.

## **YETI Dentalprodukte GmbH**

Industriestr. 3 D-78234 Engen - Germany  
Tel.: 07733-94 10 0 / Fax 07733-94 10 22