

Operating Instructions

Status of technical instructions: 05/01

1. General terms:

- 1.1. Product description:** ISO-QUICK item #550-0000
YETI LUBE item #550-0001/ #550-0006/ #550-0250
- 1.2. Delivery includes:** #550-0000 Isolating Pen (filled)
#550-0001 1 bottle/ 5500006 6 bottles of 20ml with appl. brush
#550-0250 1 refill bottle of 250 ml
- 1.3. Manufacturer's address:** YETI Dentalprodukte GmbH - Industriestrasse 3 - D-78234 Engen
phone: 0049-7733-9410-0 FAX: 0049-7733-941022
- 1.4. Storage conditions:** Store at room temperature with protective caps (ISO QUICK).
Keep out of the sun and away from direct heat.

2. Uses:

For isolating plaster, metal and wax surfaces against wax used in dental laboratories. The isolating YETI LUBE - with which the pen is filled - can be refilled at any time.

Any misuse of the pen will lead to loss of warranty and liability!

3. Caution:

Ensure that unqualified persons (e.g. children) do not use this pen. For dental technicians and authorised persons only.

4. Operating instructions:

4.1. Before use:

Pen: Take off the protection cap of the Isolating pen (end with the pointed tip).

4.2. Operation:

Apply Yeti Lube with a slight pressure of the ISO QUICK pen or with the brush and work on a suitable surface

(see point 2). The effect of isolator will stay hours after application. Apply the wax and lift off after modelling.

ISO QUICK: For isolating large surfaces, use the flat wide side of the applicator.

4.3. After use:

After application of the isolator, the protection cap must be replaced on the pen/ the bottle must be closed.

4.4. Refill:

When the isolator in the pen is used up, the ISO-QUICK pen can be refilled: pull the applicator out of the pen by means of tweezers. The tank which is now accessible, can be refilled with YETI LUBE; most suitable is the YETI LUBE refill bottle (#550-0250) equipped with an especially pointed refill tip.

5. Disposal:

When disposing of the pen, please observe procedure operating in your country. The packaging is recyclable.



Operating Instructions

Status of technical instructions: 05/01

6. Durability: In general, all material should be used within several months after it has been opened.

