

# Operating Instructions

---

Status of technical operating instructions 5/2001

## **1. General terms:**

- 1.1. Product description:** THOWAX Sculpturing wax, several colours  
VKS Sculpturing waxes for full ceramic systems  
IQ Sculpturing wax, several colours
- 1.2. Delivery form:** tin of 70g / 45 g / 20 g / bloc of 45 g / chip of 3x20 g/  
wax pearls 80g
- 1.3. Manufacturer:** YETI Dentalprodukte GmbH - Industriestraße 3  
78234 Engen/Germany  
Tel.: 0049-7733-9410-0 Fax: 0049-7733-94 10 22
- 1.4. Storage:** Store at a temperature of at least +10°C to 30°C  
Protect from UV-radiation; temperatures below 8°C, especially  
below 0°C make the wax brittle. Protect from blow or any  
impacts.

## **2. Application according to determinations:**

Our different waxes are used for sculpturing of tooth restoration in dental laboratories in wax as an initial stage for the final denture. Due to the variety of colours and different characteristics, these waxes allow to practice all the different techniques.

**Any claims on warranty or liability will not be accepted if not used according to determinations !**

## **3. Risk and danger during application:**

1. Ensure that unqualified persons do not come into contact with this product.
2. Avoid hot wax drops on skin and avoid also inhalation of wax vapours

## **4. Operating instructions:**

### **4.1. Preparation**

Open the tin.

### **4.2. Application**

Choose a suitable instrument or an electric waxer for wax sculpturing. The object you want to wax up has to be isolated. Use Yeti Lube as separating liquid. Then begin your waxing-up. Surplus wax can be scraped off.

### **4.3 After application**

Close the tin after use to protect from dust or any other pollution.

**5. Disposal:** According to current local regulations.

**6. Durability:** Consumer goods should be used rapidly after opening and used up within a few months.