

# Operating Instructions

Status of technical operating instructions 1/96

## 1. General terms:

- 1.1. Product description:**
- IQ Sticks 2,5 mm 768-2500  
IQ Sticks 3,0 mm 768-3000  
IQ Sticks 3,5 mm 768-3500  
IQ Sticks 4,0 mm 768-4000  
IQ Sticks 5,0 mm 768-5000
- IQ Triangel 4 mm 758-5000  
IQ Triangel 5 mm 759-5000
- IQ Quadro Sticks 2,5 mm 765-5000  
IQ Quadro Sticks 3,0 mm 766-5000
- 1.2. Delivery form:**
- IQ Sticks round red, extra-hard, 250 g  
IQ Triangel casting bar red, extra-hard, packet 50 pc.  
IQ Quadro sticks green, extra-hard, 50 g
- 1.3. Manufacturer:**
- YETI Dentalprodukte GmbH - Industriestraße 3 - D-78234 Engen/Germany  
Tel.: 0049/7733-9410-0 FAX: 0049/7733-9410-22
- 1.4. Storage:**
- Store at a temperature of at least +10°C to 30°C  
Protect from UV-radiation; temperatures below 8°C, especially below 0°C  
make the wax brittle. Protect from blow or any impacts.

## 2. Application according to determinations:

Our Wax Sticks and Wax Triangel are for the use of dental laboratories, especially for the casting technique. The shape of different Wax Sticks can be changed easily through warming up. Sticks and Triangels should cool down again before waxing-up.

The middle bar of the Triangel can be easily bend open by warming it up .

**Any claims on warranty or liability will not be accepted if not used according to determinations !**

## 3. Risk and danger during application:

1. Ensure that unqualified persons do not come into contact with this product.
2. Avoid hot wax drops on skin and avoid also inhalation of wax vapours.

## 4. Operating instructions:

### **4.1. Preparation**

Open the packet.

### **4.2. Application**

Shorten the Wax Stick or Triangel to the desired size/lenght. Wax up the casting system with warm wax - we recommend to choose same sort of wax. After cooling-down Wax-Deblubblizer ( art. no. 142-0000 ) can be applied, then embed.

### **4.3 After application**

Put back rests into packet and close this after use to protect from dust or any other pollution.

**5. Disposal:** According to current local regulations.

**6. Durability:** Consumer goods should be used rapid after opening and used up within a few months.