



1. General information:

- 1.1. Article:** PRECI TRAY lightcuring material art.no. 890-1000 pink
art.no. 890-1010 white
art.no. 890-1020 blue
- 1.2. Delivery form:** 1 light-protected box with 50 wafers (upper jaw form)
- 1.3. Manufacturer:** YETI Dentalprodukte - Industriestraße 3 - D-78234 Engen
Tel.: 0049-7733-94100 Fax: 0049-7733-941022
- 1.4. Storage:**
- store cool, light protected and frost-free
 - storage temperature not less than -10°C and not more than +50°C

2. Use according to determinations:

This synthetic material can be easily formed at room temperature and daylight to obtain base plates and forms for reproductions used within the dental field.

This material shall be used inside the mouth for short-term only and it is not recommended for any other intraoral use. The individual form for reproduction should be formed on the plaster modelation and hardened with a suitable halogen- or UV-source of light. within 3 - 5 minutes (wavelength 350-400 nm). Longer polymerizations improves its physical characteristics.

Any claims on warranty or liability will not be accepted if product is not used according to determinations.

3. Risks or dangers during application:

- Ensure that no unqualified persons (e.g. children) come into contact with this product. Only dental technicians or other instructed and trained persons should work with this material. Operating instructions must be read and followed before application.

4. Operating instructions:

4.1. Preparation:

- Apply a separating layer onto the plaster modelation (spacer function), or any suitable isolating material for the separation of plaster from synthetics.

4.2. Application:

1. Open the packet on the upper top.
2. Take the required wafer and remove the separating layer.
3. Shape into the desired form and remove edges before hardening.
4. A grip can be added before or after hardening procedure.
5. The sticky inhibition layer can be easily removed using alcohol.

4.3. After application:

- Store reusable rests inside the closed box and protect from light and dust.

5. Disposal: Follow the current local regulations.

6. Guarantee: Durability last for 24 month after opening the original box.